




**Ultimated Operability!**

# Suction Toothbrush Suction Sponge

Smooth suction and mouth cleaning with one hand !

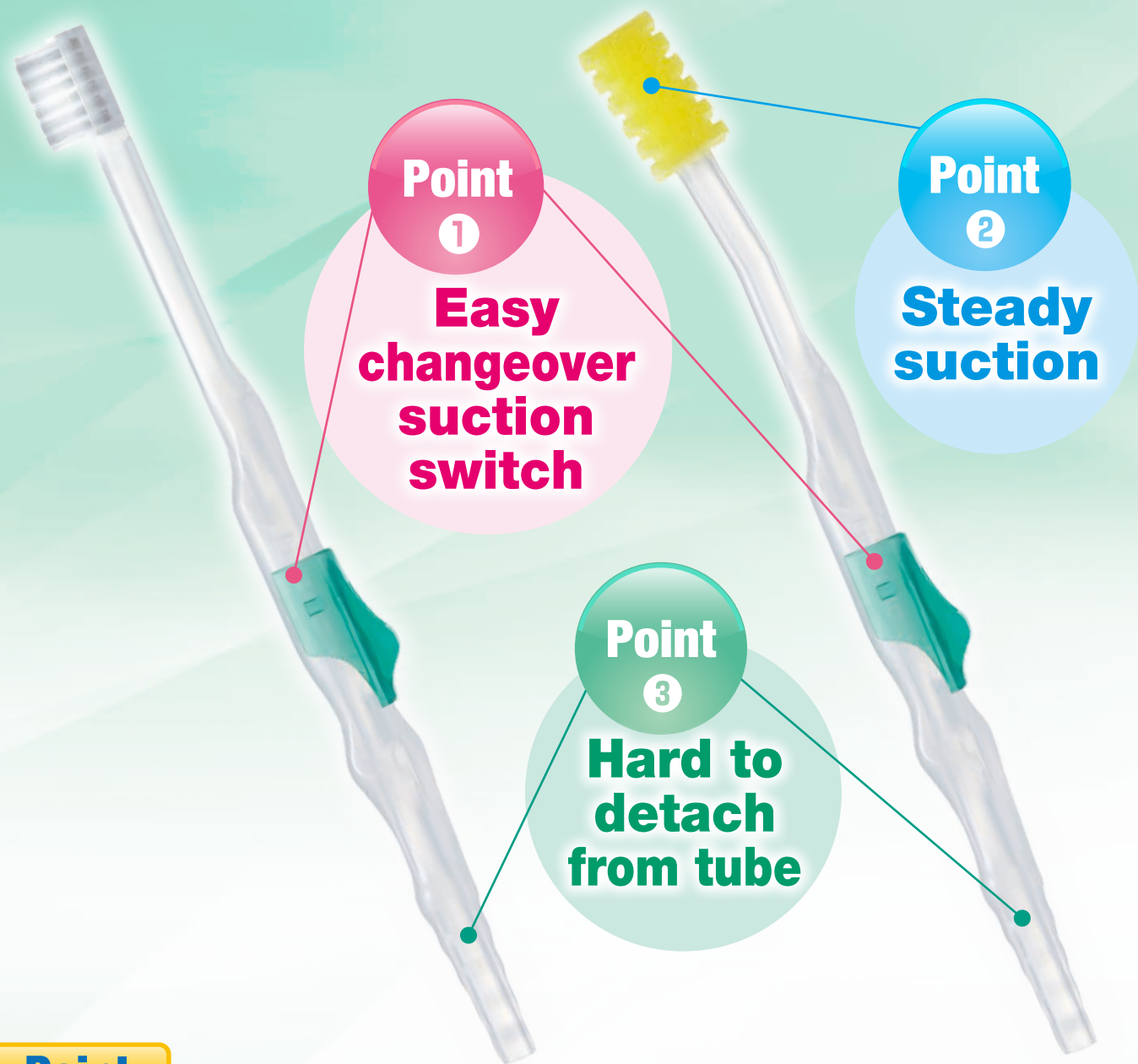

**Point**

- ① Easy to hold and change function using the suction on/off switch!
- ② The wide suction aperture ensure the suction of even viscous debris !
- ③ The shape provides a snug fit for the suction tube, holding it firmly and ease of use!


**KAWAMOTO CORPORATION**  
OSAKA · JAPAN

## How to Use

### ① Suction Toothbrush / Suction Sponge


Connect to the suction tube.

Suitable inner diameter of the tube is from 5 to 8 mm.

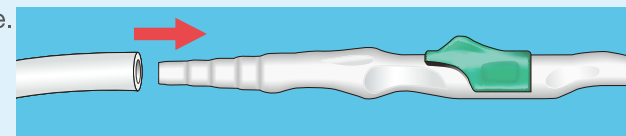


### ② Suction Toothbrush

While the switch is off, the brush part absorbs small amounts of water. Gently clear the water before brushing the teeth.

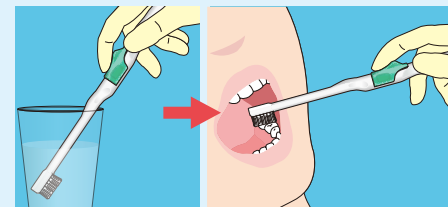


### ② Suction Sponge

While the switch is off, the sponge absorbs small amounts of water. Squeeze out the sponge part thoroughly before cleaning oral mucous membranes (upper jaws, gums), the tongue, or other sites.

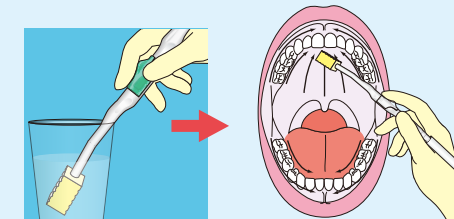

### ③ Suction Toothbrush / Suction Sponge

When moisture on the toothbrush and sponge becomes excessive during cleaning, switch on suction by sliding the switch towards you to suck up aspirate.

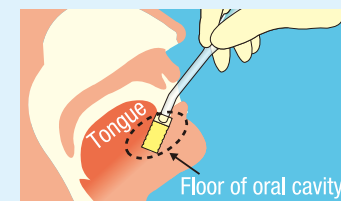

Use the tip of the toothbrush or sponge to suck up saliva from the floor of the oral cavity where it is prone to accumulate.

**[Important notes]**

- Large debris in the oral cavity may clog the suction aperture, handle, or tube. Remove this first and only suck up moisture.
- This product is disposable.

## Size

Suction Toothbrush	Suction Sponge
Tip ← 10 →	Tip ← 17 →
14	15
Cleaning surface	Cleaning surface
19 (mm)	25 (mm)



Item	Spec.	Packing
MOUTH PURE SUCTION TOOTHBRUSH	1 pc/pack	1 pc/pack, 20 packs/box, 12 boxes/carton
MOUTH PURE SUCTION SPONGE	5 pcs/pack	5 pcs/pack, 20 packs/bag, 6 bags/carton
MOUTH PURE SUCTION SPONGE	50 pcs/box	50 pcs/pack, 12 boxes/carton

**KAWAMOTO CORPORATION**

6-4 TANIMACHI 2-CHOME, CHUOKU OSAKA 540-0012, JAPAN. TEL : 81-6-6943-8991 FAX : 81-6-6943-9081  
http://www.kawamoto-sangyo.co.jp. E-mail : kbm@kawamoto-sangyo.co.jp.

# Feature

# Smooth suction and mouth cleaning with one hand !


## 1 Easy change of suction on/off by switch

For ease of use, you can switch suction on and off by switch without having to keep your finger in a fixed position.

[Finger control type]


[Switch type] (ON)

(OFF)

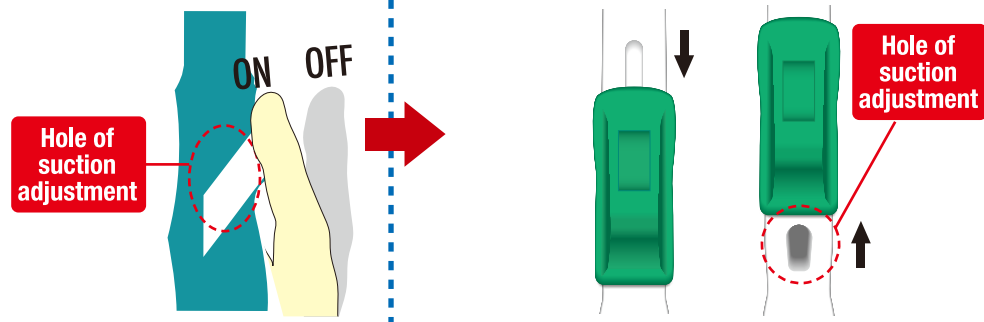


## 2 Thorough suction while cleaning

There is a suction aperture at both the tip and the cleaning surface, making it easy to suck up moisture and cleaned debris that has accumulated at the back of the mouth.

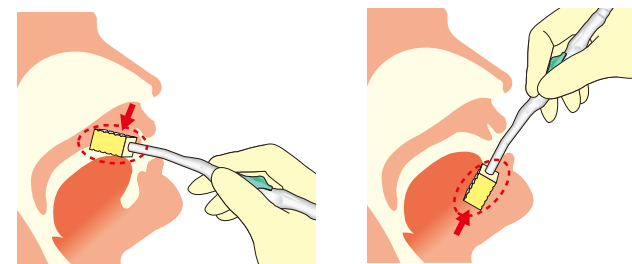

## 3 Steady suction even for viscous debris

The wide suction aperture ensure the suction of even viscous debris.

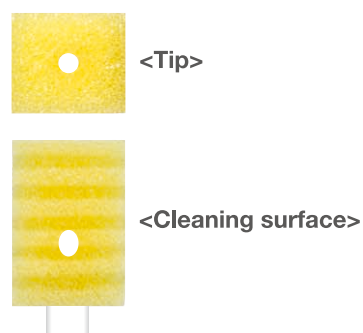

## MOUTH PURE SUCTION TOOTHBRUSH



Feature of Toothbrush



(Actual size)

## MOUTH PURE SUCTION SPONGE



Feature of Sponge



(Actual size)

## 4 A soft brush that gives a thorough polish

Using a toothbrush with good cleaning ability that does not easily damage mucous membranes.

Product	Gentleness to mucous membranes (mucous membrane damage test)	Cleaning effect (cleaning test)
MOUTH PURE SUCTION TOOTHBRUSH	◎	◎
General toothbrush (Soft type)	△	△
General toothbrush (Normal type)	△	◎
General toothbrush (Hard type)	×	◎

(In-house research)


## 5 Aspirate is easy to see

The handle is semi-transparent, enabling you to check on aspirate.




## 6 Fits perfectly into even curved upper jaws

The cleaning surface easily follows even curved upper jaws and the inside of cheeks.

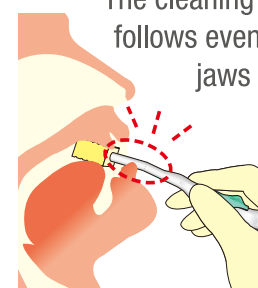

## 7 Hard to detach from tube and easy to use

The waved shape provides a snug fit for the suction tube, holding it firmly and ease of use.

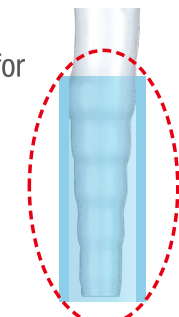

MOUTH PURE SUCTION TOOTHBRUSH (1 pc/pack)



MOUTH PURE SUCTION SPONGE (5 pcs/pack)



MOUTH PURE SUCTION SPONGE (50 pcs/box)